ic! berlin

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EN SPECTACLE FRAMES LABELING

Field of use

Spectacle frames are medical devices that are principally intended for use in connection with ophthalmic prescription lenses for correcting visual problems and they are supplied with their own case. Spectacle frames comply with the requirements set out in standard EN ISO 12870:2018.

Warning

Adverse contact reactions to materials and treatments used for the frames in particularly sensitive or allergic subjects cannot be ruled out. The frame comes with clear lenses that are not intended for use but are designed to only prevent frame warping and allow for a more realistic try-on. The assembly of spectacle frames with diopter ophthalmic lenses may only be performed by qualified professionals. All other uses are prohibited. **Should users experience any serious repercussions related to the use of this medical device, they are required to report it to the manufacturer and to the national Competent Authority.** This Spectacle frames labeling contains important information about use, care and cleaning of Spectacle frames. Please read it carefully before using the product. The manufacturer will not accept any liability/warranty claim for any damage caused by the improper use and/or by lack of/incorrect maintenance of this product.

Storage and Cleaning

Store the Spectacle frames in their case at a temperature between -10°C and +35°C. Keep away from direct sunlight and avoid exposure to high temperatures, such as the heat in a car without conditioning (e.g. car dashboard), because high temperatures may alter the features of the glasses. In case of contact with agents such as salt water (seawater), swimming pool water, chemical products such as make-up, hairspray, sun lotions, and insect repellents, we recommend that you immediately remove such agents using a soft damp cloth. In case of wear and tear (e.g. scratched, opaque), we recommend the use of original replacement spare parts. Use a damp cloth and mild soap to clean the glasses, then dry them with a soft, clean cloth. Do not use solvents (e.g. alcohol, acetone) or aggressive detergents which may alter the features of the glasses. Do not use the device if it clearly appears damaged. Do not discard the product, the case and relevant packaging in the environment; dispose of the product, case and associated packaging in accordance with current local regulations.

Marking

The name of the model is on the inside of the temple of the glasses. The date of manufacture is indicated in the format "YYYY-MM-DD" on a label placed on the individual packaging of the glasses. The EU declaration of conformity of Spectacle frames and copy of the Manufacturer information booklet are available on the following website: <https://www.ic-berlin.com>.

Reporting/Contact

Complaints and anomalies related to this product and/or of its case must be reported to the dealer/store where you purchased this product or must be addressed to: **info@ic-berlin.de**. In case of safety-related complaints with this product and/or of its case, please report them by e-mail to **compliance@ic-berlin.de**.

Serial number

The serial number is indicated on the inside of the temple of the glasses and on a label placed on the individual packaging after the symbol "SN."

Our products are protected by the following patents:

Models with 6 mm and 10 mm temples:

EP2614402, US patent no. 9069190, Chinese invention patent 中国发明专利号 ZL

201180004401.9, Patent 1168659HK, JP 5770274, KR 10-1308221, Patent SG

184551, Thai patent no. 51009, TW I 461780

Models with 2.5-3.5 mm temples (Silk collection):

US patent no. 9,016,856, TW patent no. I 497148 B

Models with acetate fronts:

EP 2140302, US patent no. 8042939, Chinese invention patent 中国发明专利号

ZL200880012743.3, JP 5047354, KR 10-1183333

Titanium models: German patent no. 10 2015 000 038

FLEXARBON models: German patents no. 10 2014 100 201.6, 10 2016 000 085.6

FLEXARBON is a registered trademark of ic! berlin.

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ic! berlin GmbH
UK Rep Med Limited